

Liko Golvo Training Checklist

Training Date: _____

Employee Name: _____ EID #: _____ Home Department: _____

Trainer's Name: _____ Trainer's Role: _____

Competency Subject	Competency Detail	Acceptable response	If unacceptable response, educate/coach on	Scoring	
				Competent, No Education Required	Education/Coaching Required
Equipment Features	Weight capacity	440 lbs. max	Where to locate the weight capacity sticker on the lift		
	Battery charging	Demonstrates knowledge of where the battery plug in cords are located, and how to engage/disengage the emergency power button	Where the battery plug in cords are located and why it's important to charge regularly		
	Changing out the battery	Demonstrates knowledge of how to remove and attach the battery from the machine	Manufacturer's instructions on removing/attaching the battery		
	Emergency lowering	Demonstrates knowledge of how to operate the emergency lowering features of the lift	Manufacturer's instructions on emergency lowering features		
Sling Compatibility	Sling options with this lift	Repo sheet, High back seated sling, Multistrap, Walking vest	Where to find the sling information in their unit's plan		
	Changing out sling bars	2-point bar, and 4-point add-on (two extra two-point bars)	Where to find the lift attachment information in their unit's plan		
Equipment Use Cases	Repositioning using the repo sheet	Correctly attaches all the repo sheet loops to the lift and demonstrates a boost	Manufacturer's instructions for the repo sheet		
	Turning using the repo sheet	Correctly attaches desired loops on one side of the repo sheet and demonstrates a turn			
	Lifting using the highback seated sling	Correctly attaches the sling to the lift, remembering to cross the legs, and demonstrates lifting and transferring to a chair	Manufacturer's instructions for the highback seated sling		
	Turning using the multistrap	Correctly anchors two multistrap to the bed frame, slides the straps under the desired point(s) of body control, and demonstrates a patient turn and hold in place	A walk through of the "acceptable response"		
	Lifting limbs using the multistrap	Correctly attaches the multistrap to the lift, ensuring that the sling position on the patient allows for the desired limb angle/positioning			
	Stabilizing legs for catheter placement using the multistrap	Correctly anchors two multistrap to the bed frame on each side of the bed, slides the strap under each thigh, and demonstrates stabilization of the patient's legs			
	Placing/removing a bedpan using a multistrap	Correctly slides the multistrap under the patient's lumbar/sacral area, secures the loops to the lift and demonstrates lifting the patient's pelvic area for placement/removal of bedpan			
	Ambulation using the walking vest	Correctly places the sling on the patient and ensures a secure fitting, connects the sling to the lift, and demonstrates ambulation assistance	Manufacturer's instructions for the walking vest		

Employee's Signature

Trainer's Signature

